

# CHAM-MATE

**Topster**

## AUTOCLAVE CHAMBER & PIPING CLEANER

### FEATURES:

- NON TOXIC, NON POLLUTING CLEANING POWDER.
- CLEANS THE CHAMBER, PIPING AND WATER RESERVOIR.
- KEEPS THE AUTOCLAVE CLEAN AND FREE OF WATER DEPOSITS BUILT AROUND THE CHAMBER, PIPING SYSTEM, AND WATER RESERVOIR.
- SPECIAL FORMULATION FOR STEAM STERILIZERS.
- CLEANS SURFACE OF STAINLESS STEEL, COPPER, SILICON RUBBER, PLASTIC, ...ETC.

**PLEASE NOTE : FOR EXTERNAL USE ONLY - NOT EDIBLE**



### CHAM-MATE clean procedures:

Basically CHAM-MATE is functioned in water.

1. For **Class-B vacuum functioned autoclaves**, or **autoclaves with independent steam generator**, the **CHAM-MATE** must be mixed with water, first. (1 bag mixed into 2,000cc water ), And then, pouring the mixed **CHAM-MATE** into the chamber, and start working cycle.
  - 1-1. For **Class-S no vacuum autoclaves (with auto-add water function)**, put the **CHAM-MATE** powder into the chamber directly, and start the working cycle.
  - 1-2. For **Class-N (manual add water function)**, put the **CHAM-MATE** powder into the chamber Directly. And then, pouring water into the chamber, and start working cycle.

**PLEASE NOTE: Running one sterilization cycle, ( without the dry function ) .**

2. Drain off all the balance water from the chamber and water reservoir.
3. Open the door, wipe the chamber and the door with a wet, non-lint cloth.

**PS: Use one more bag of CHAM-MATE and recycle again as above procedures, if there are dregs in the chamber still.**

**NOTE: "For chamber and piping system clean only."**

### CHAM-MATE quantity to be used:

Chamber size under 30 liter---1 bag cham-mate.  
Chamber size 30~100 liter-----2 bags cham-mate.  
Chamber size over 100 liter----1 bag more per 100 liter more.

**STURDY INDUSTRIAL CO.,LTD.**

SO-9001 & ISO-13485 CERTIFICATED FIRM