

SPECIFICATION:

• LC-01: Standard port x 3

Rotation port x 1

• LC-01A: Standard port x 2

Rotation port x 2

• LC-01B: Standard port x 1

Rotation port x 3

**Standard port: High speed; Rotation port: Low speed

Power Supply: 110 - 120V or 220 - 240V; 50/60Hz • Air Pressure: 0.35 - 0.6 Mpa (3.5 - 6 kgf/cm²)

• Weight: 7.3 kg

• Dimension: 360mm (W) x 290mm (D) x 340mm (H) 350 ml (Oil 0.35 ml per handpiece) Oil Tank Capacity:

Standard Accessories:

1. Power Cord x 1

2. Maintenance Oil x 1 (1 liter bottle)

3. Oil Replenishment Funnel x 1

4. Fuse x 2

5. Air Tube x 1

6. Y-type Quick-connecting Joint x 1

7. Filter Sponge x 6 (Pack of 2)

8. Magnet x 1

9. Test Bur x 2

10. O-ring x 2





STURDY INDUSTRIAL CO., LTD.
168, SEC. 1, ZHONGXING RD., WUGU DISTRICT, NEW TAIPEI CITY 24872, TAIWAN TEL: 886-2-29815245 • FAX: 886-2-29848208 E-MAIL: foreign@sturdy.com.tw

0902-DM-3002



HANDPIECE CLEANER



Best Answer to Longer Lifespan of Your Handpieces!

Handpiece Cleaning, Lubrication, and Maintenance in One Cycle



FULLY AUTOMATIC MICROPROCESSOR CONTROLLED SYSTEM 4 HANDPIECES PER CYCLE - FAST and EFFICIENT

FEATURE:

- The perfect solution for fast and efficient maintenance for different brands of high and low speed handpieces.
- The automatic rotation function during the cycle guarantees that each handpiece is cleaned and lubricated thoroughly.
- 3. The front cover can be completely removed by the touch of a button for easy cleaning.
- 4. Level indicator of the oil tank helps users to check the volume of the oil easily.



5. Press the "AIR" button; it will activate the air compressor to flush out all residual solution.



6. Filter sponges can absorb the spray mist. The sponges are easy to be replaced.



 Different brands of all E-Type low speed handpieces can be attached to the connector directly.
 With the adapter, all different brands of handpieces can be attached to the connector for direct use.

- 8. Simple and Easy-to-Understand Control Panel:
- Cycle "S" (approx. 35 seconds): Short cycle for high speed handpieces
- Cycle "M" (approx. 45 seconds):
 Medium cycle for low speed handpieces
- Cycle "L" (approx. 50 seconds): Long cycle for all handpieces
- 9. Error Indicator:

The indicator light will illuminate if the door isn't closed completely. You will not be allowed to operate the cleaner in the meantime.



10. Complete Alarm:

A buzzer will sound when the cycle is complete.

